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K-jump Health Co., Ltd.,
No.56, Wu Kung 5th Rd.
New Taipei Industrial Park,
New Taipei City, 24890 Taiwan

Importeur & klantendienst
SCALA Electronic GmbH, Ruhlsdorfer Straße 95
14532 Stahnsdorf, Germany

Polygreen Germany GmbH
Ruhlsdorfer Str. 95
14532 Stahnsdorf, Germany

scala®

Multifunctional Wrist Blood Pressure Monitor Model SC 7400 NFC

CE 0197

 **Instructions for use**

Contents

| | |
|--|----|
| Special features of the SC 7400 NFC | 3 |
| Precautions | 3 |
| Product description and intended use | 5 |
| About blood pressure | 5 |
| The product | 8 |
| The LC display | 8 |
| Explanation of the symbols used | 9 |
| Preparations for start-up | 10 |
| Applying the cuff and measurement position ... | 11 |
| Performing the measurement | 12 |
| The memory function | 13 |
| Care and maintenance..... | 14 |
| Cleaning | 14 |
| Regular re-calibration | 14 |
| Technical specifications | 15 |
| Warranty | 15 |
| Disposal | 16 |

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Specific features of the SC 7400 NFC

- The product uses most advanced technology for reliability in blood pressure measurement. The blood pressure is measured while the cuff is inflating (smart logic).
- No need ever to worry about batteries. The product has a built-in lithium power source.
- Data can be transferred to a smart phone/tablet by NFC (*if an NFC capable smart phone/tablet is used*). For this, install the free BPM OUCare app.
- At rest, the product displays the time and the room temperature.

Precautions

1. Do not overinflate the cuff because the pressure on the arm will be painful.
2. Open the velcro fastener when the inflation pressure is higher than 300 mmHg.
3. Keep the blood pressure monitor out of reach of children to avoid misuse.
4. The BP monitor is not a substitute to your visit to a doctor. Diagnosing and treating a disease and prescribing drugs are the doctor's job.
5. The BP monitor cannot work as it normally should when it is used outside the specified temperature and humidity ranges (see Technical Specifications).

6. Do not eat or smoke and do not move during the measurement. Do not drink coffee or alcohol half an hour before a measurement.
 7. Relax 5 or 10 minutes before the measurement.
 8. Wait 3 to 5 minutes before making the next measurement.
 9. Always make the measurement at room temperature. Do not use the product near a microwave oven.
 10. Remember that your blood pressure is subject to natural variation. The pressure is highest under stress and lowest when you sleep.
 11. Ask your doctor to explain your state of health and the blood pressure range before you start making measurements at home.
 12. If you suffer from disease, such as: arteriosclerosis, diabetes, liver or kidney problems, circulatory disorder or hypertension, consult your doctor before using the blood pressure monitor.
 13. During the measurement, the device displays possible cardiac arrhythmia such as arterial or ventricular extrasystoles or atrial fibrillation.
 14. If you do not tolerate the measurement for any reason, press the ON/OFF button to switch off the device and deflate the cuff immediately.
 15. Handicapped individuals should not themselves measure their blood pressure.
 16. Do not make changes to the device or the cuff. Stop using the device when you believe that it is defective.
- GB-4

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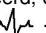
NL-12

PLEASE NOTE:

If you do not use the product for a longer time, recharge the battery every 3 to 6 months to prevent the deep discharge of the LiPo battery.

Product description and intended use

The blood pressure monitor is an electronic device for the non-invasive self-measurement of the human blood pressure and the heart rate in adult persons by the oscillometric measuring method on the left exposed wrist of an adult person at home at room temperature. Handicapped persons themselves cannot use the blood pressure monitor; they need the assistance of a person without a handicap. The measuring values can be transferred to a smart phone or tablet computer by NFC. For that, install the free BPM OUCare app.

About blood pressure

What is blood pressure?

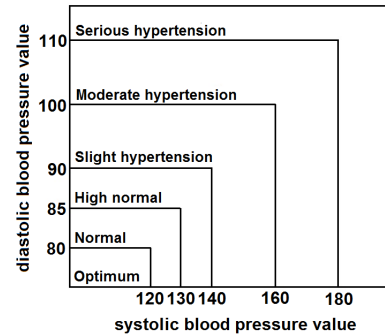
The blood pressure is generated by the force of the blood flow against the arterial walls after the heart has pumped the blood in the circulatory system.

What are the systolic and the diastolic pressure?

The systolic pressure is the highest pressure while the heart contracts; the diastolic pressure is the pressure while the heart is at rest.

GB-5

The WHO (World Health Organization) defined the blood pressure levels as follows in 1999:

What is low blood pressure?

Low blood pressure causes several uncomfortable symptoms such as dizziness or drowsiness but is less harmful than high blood pressure.

Blood pressure variations

The blood pressure in humans varies during the day (see chart). The causes are:

Eating, bathing, doing sport, smoking, drinking alcohol, physical and mental stress, breathing, conversation, exercise, changes of temperature and humidity, etc.

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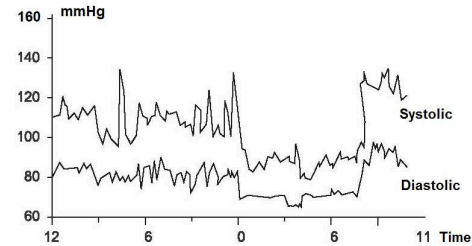
NL-10

Possible variations of the blood pressure during a day:

How to obtain reliable measurement values?

For reliable measurement values, please follow our recommendations:

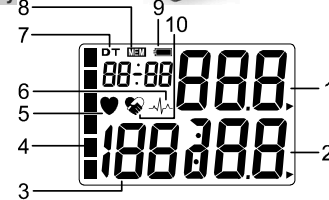
1. Always make the measurement at the same time and under the same conditions.
2. Do not eat, smoke or move during the measurement. Do not drink coffee or consume alcohol within 30 minutes before the measurement.
3. Remove tight clothing from your wrist and make sure that the area for the cuff is free.
4. Sit upright at the table and relax for about 5 minutes to ensure that the measurement is made in a steady atmosphere.
5. If you want to repeat the measurement, wait at least 5 minutes.

GB-7

The product

The LC display


- 1 Systolic blood pressure
- 2 Diastolic blood pressure
- 3 Heart rate display
- 4 WHO display
- 5 Heart rate detected
- 6 Indicates cardiac arrhythmia
- 7 Date/time
- 8 Memory
- 9 Battery level indicator
- 10 Connection with smart phone

The delivery includes: the blood pressure monitor, a storage bag or box, the micro USB cable and the instruction manual.
GB-8

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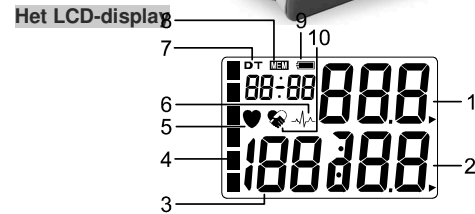Meaning of the symbols used

| | | | |
|----------------|--------------------------------|----------------|---------------------------------|
| SYS | Systolic blood pressure value | CON | Connect product to smart phone |
| DIA | Diastolic blood pressure value | IN | Measuring values are taken over |
| | Indicates cardiac arrhythmia | | Disposal information |
| | Read the instruction manual | | Interference source nearby |
| | Manufacturer | LOT MMY | Manufacturing date (month/year) |
| | BF usage part | P | Pump running |
| | Protect from moisture | P Err | Pump defective |
| CE 0197 | Number of the Notified Body | UU Err | Error when you are moving |
| | ON/OFF button | LL Err | No pulse detected |
| SN | Product number | rrErr | No plausible measuring value |
| | Charge battery | HI | Cuff pressure over 300 mmHg |
| EC REP | European representative | | |

Preparations for start-up

Charging the battery

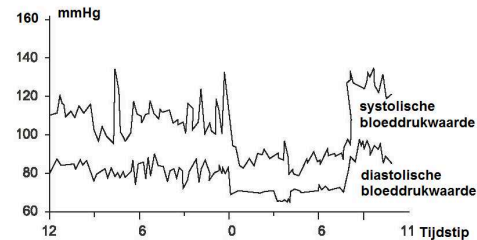
1. The symbol "  " on the LC display indicates that the battery is depleted. Connect the product to a USB port of a PC or a USB charger with the micro USB cable.
2. The battery symbol in the LC display flashes or the battery charge is indicated in per cent.
3. The battery symbol goes out when the battery is full.

Setting the timer and the temperature display

1. Open the lock of the ON/OFF button (see figure on page 8).
2. With the device switched off, press and hold the MEMORY button until the flashing year is displayed. Set the year with the ON/OFF button.
3. Press the MEMORY button again to enter the setting of the month, day, hours and minutes.
4. Then "CL" is displayed. Press the ON/OFF button "on" for constant display of the time or "off" to hide the time. Up to 4 alarm times (AL1 - AL4) can be set. When an alarm is set, the alarm time is displayed.
5. Press the MEMORY button once more to enter the temperature display. Toggle between °C, °F, and no temperature display (oFF) with the ON/OFF button. Complete all settings by pressing the MEMORY button. The alarm times are not activated by locking the ON/OFF button.

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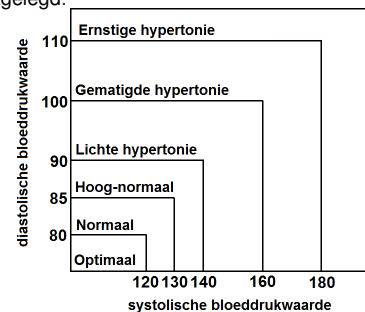
NL-6

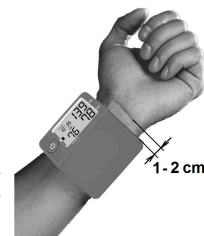
Putting on the cuff and measurement position

1. Check to be sure that the circumference of your wrist fits the cuff.

2. Place the cuff around your exposed **left** wrist. Interfering clothes and jewellery should be removed beforehand. The top edge of the monitor should be 1 or 2 cm away from the wrist.

3. Close the cuff. Do not put on the cuff very tightly.

4. Sit upright on a chair at the table. Put your feet flat on the floor. Relax.

5. Make sure that the cuff is on an imaginary line with the heart. (See opposite figure).

6. Start the measurement with the ON/OFF button. Do not move or speak.

7. If you cannot make the measurement on the left wrist, use the right wrist.

GB-11

Performing the measurement

1. Open the lock of the ON/OFF button (see figure on page 8).
2. Select a user with the User button.
3. When you have placed the cuff correctly around the wrist, press the ON/OFF button to start the automatic measurement.
4. At the end of the measurement, the systolic and the diastolic pressure and the heart rate are displayed.
5. WHO blood pressure rating

Date: 29/12(2007)

| Class | Systolic / mmHg | Diastolic / mmHg | Colour bar |
|-----------------------|-----------------|------------------|---------------|
| Optimum | < 120 | < 80 | 6 green LEDs |
| Normal | < 130 | < 85 | 3 green LEDs |
| High normal | 130 - 139 | 85 - 89 | 6 yellow LEDs |
| Slight hypertension | 140 - 159 | 90 - 99 | 2 red LEDs |
| Moderate hypertension | 160 - 179 | 100 - 109 | 4 red LEDs |
| Serious hypertension | ≥ 180 | ≥ 110 | 6 red LEDs |

According to WHO, 140/90 mmH is hypertension.

6. If irregular heart rate is detected, the symbol "⚡" is displayed. Cardiac arrhythmia is indicated when the period between two heart beats is 5/3 the time of the average heart beat and this abnormality occurs three times during a measurement. Consult a doctor when cardiac arrhythmia occurs frequently.

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7. When the device is switched off, the measuring values are stored.
8. Close the lock of the ON/OFF button.

The memory function

1. Open the lock of the ON/OFF button.
2. With the User button, go to the memory of the user whose value you want to display.
3. Display the measuring values in the memory with the MEMORY button.
4. The value stored last in the memory is displayed (it is placed at the highest memory location).

Deleting a value from the memory

With the MEMORY button, go to the blood pressure value which you want to delete. Instantly press and hold (at least 4 seconds!!!) the ON/OFF button and the display will show: dEL (delete) and the number of the memory location. To delete the value, press the ON/OFF button instantly.

Deleting all values in the memory

With the MEMORY button, go to any blood pressure value. Instantly press and hold (at least 4 seconds!!!) the ON/OFF button and the display will show: dEL (delete) and the number of the memory location. Instantly press the MEMORY button and dEL (delete) and ALL (for all memory locations) will be displayed.

Instantly press the ON/OFF button and all values are deleted. Time and date are not deleted.

Care and maintenance

1. Do not drop the device; it is not resistant to impact.
2. Do not remove or modify the device or the cuff. Do not twist the cuff.
3. Avoid direct sunshine, high temperatures, moisture, dust and dirt.
4. Do not use thinner, petroleum ether or other aggressive cleaner.
5. Do not wash the cuff or immerse the cuff in water.

Cleaning

1. Clean the device with a cloth moistened with water or a neutral cleaner. Wipe it dry after cleaning.
2. Stains on the cuff can be removed carefully with a moist cloth and soap solution.

Regular recalibration

No checks of the measuring accuracy of the SC 7400 NFC are normally required. Devices used in medical practice in Germany must be checked after 2 years according to applicable legislation. Information can be obtained from our customer service.

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Technical specifications

| | |
|-----------------------|---|
| Measuring range | Pressure: 20 ~ 300 mmHg Heart rate: 40 ~ 200 bpm |
| Accuracy | Pressure: 3 mmHg Heart rate: $\pm 5\%$ of the measuring value |
| Measuring method: | Oscillometric measurement (smart logic) |
| Display | LC display |
| Memor | 2x20 sets |
| Battery capacity | approx. 80 measurements |
| Weight | approx. 82 g |
| Cuff size | 135 - 220 mm |
| Operating environment | 5 ~ 40°C, 15 ~ 90% RH |
| Storage conditions | -20 ~ 55 °C, <93% RH |
| Operating voltage | Lilo battery 450 mAh, micro USB cable |
| Automatic Off: | after approx 150 s |
| IP classification | IP20 (the product is not water-proof) |

Specifications may change as technical revisions take place.

Warranty

We grant a warranty of two years for the blood pressure monitor. The cash voucher is your warranty certificate. The product will be repaired free during the warranty period. You can also have your BP monitor repaired by our customer service after the warranty period. We will provide you with a free cost estimate for the repair.

The warranty does not cover damage due to improper handling, accident, non-compliance with the instructions for use or unauthorized modifications of the product.

Disposal

Defective blood pressure monitors or monitors at the end of their life may contain material that might be a hazard to the environment. Dispose of the blood pressure monitor as prescribed, i.e., at a collection point for electrical scrap.

The disposal of batteries is regulated. Spent batteries are collected at collection points provided by battery dealers and the communities.

K-jump Health Co., Ltd.,
No.56, Wu Kung 5th Rd.
New Taipei Industrial Park,
New Taipei City, 24890 Taiwan

Importer & after-sales service
SCALA Electronic GmbH
Ruhlsdorfer Straße 95
14532 Stahnsdorf, Germany

Polygreen Germany GmbH
Ruhlsdorfer Str. 95
14532 Stahnsdorf, Germany

scala®

Multifunctionele polsbloeddrukmeter SC 7400 NFC

CE 0197

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